


# Ceiling light Amber LED Light Parts list

02KD01-00A

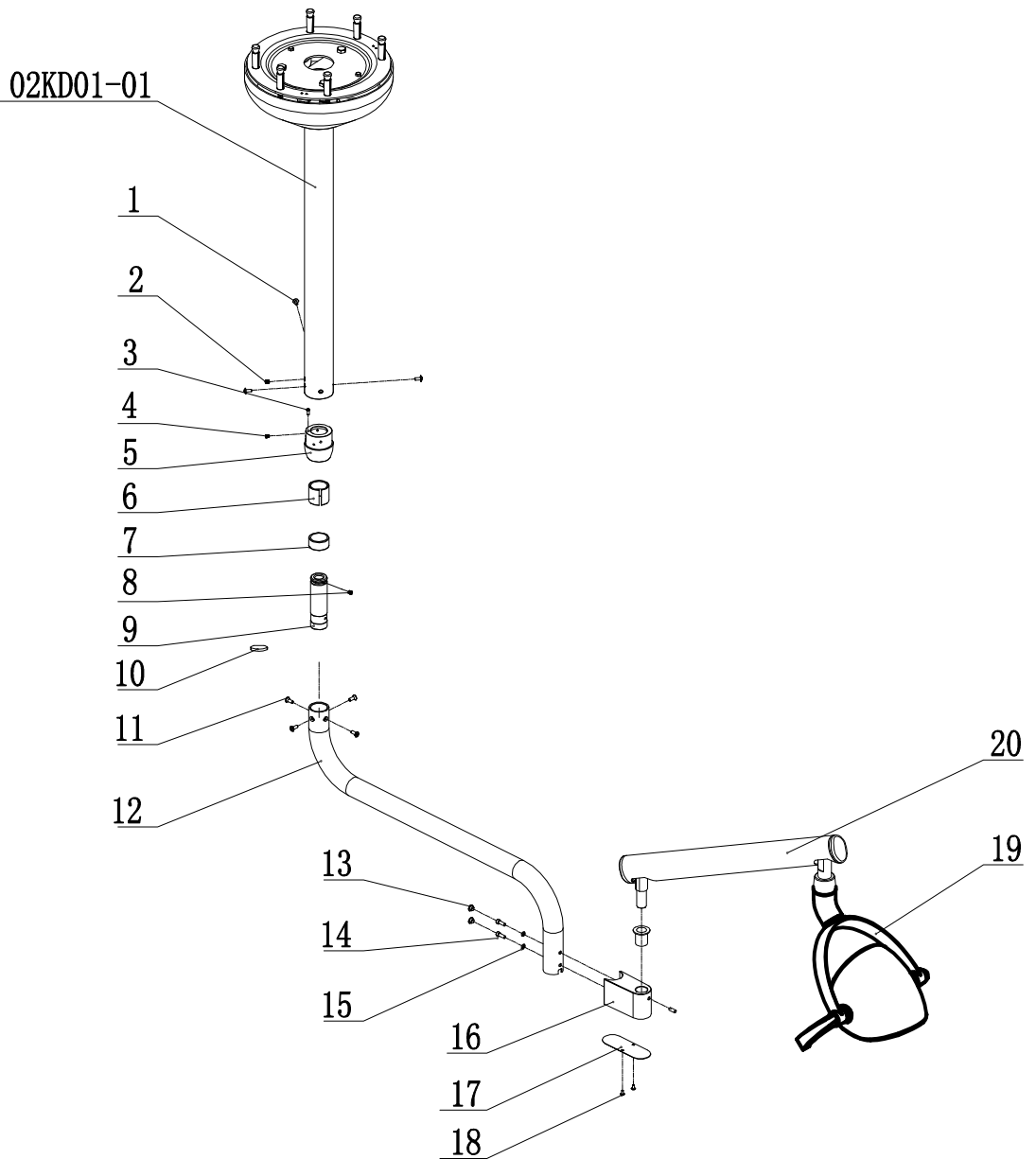




# Ceiling light Amber LED Light Parts list

02KD01-00A

No.	Part number	Name	Number	Remark
1	8022876A	M5*8 Stainless steel screw	4	
2	8013801A	M6*8 set bolt (black)	1	
3	8001684A	M4*16 Stainless steel screw	1	
4	8021175A	M5*6 Stainless steel screw	1	
5	8017785B	ceiling light	1	
6	8022522A	copper sleeve	1	
7	8017785B	HK3520needle bearing	1	
8	8022522A	M6*8 set bolt (black)	1	
9	8017779C	light arm connecting shaft	1	
10	8022528A	DL-2 Rotation limit	1	
11	8022876A	M5x8 Stainless steel screw	4	
12	8017791B	adjustable light arm II-balance arm (1.72Kg)	1	
13	8023527A	DL-2 ceiling light rotation arm	1	
14	8011491A	glass cuspidor silicone decoration cover	2	
15	8001738A	M6*16 Stainless steel screw	2	
16	8001564A	φ6 spring pad	2	
17	8023530A	ceiling light balance arm fixed seat	1	
18	8023533A	ceiling light balance arm fixed seat seal board	1	
19	8020617A	Amber LED dental light (ADS)	1	
20	8014164A	M6*5 alloy steel set screw	1	

# Ceiling light Amber LED Light Parts list

02KD01-01A

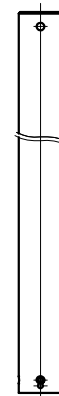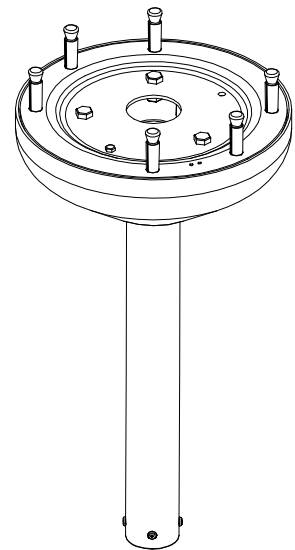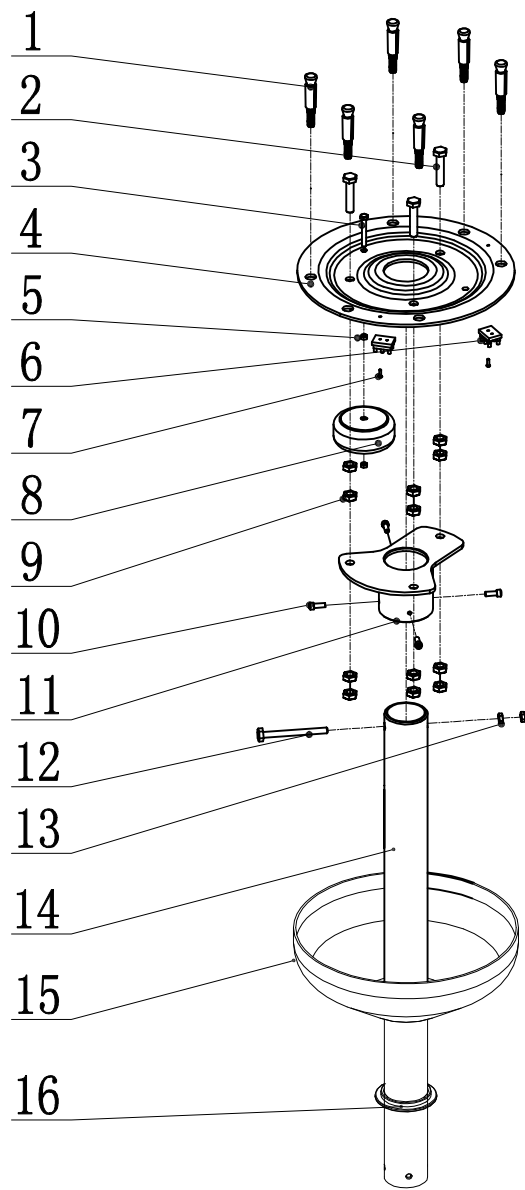

8'



9'



10'

## Ceiling light Amber LED Light Parts list

02KD01-01A

No.	Part number	Name	Number	Remark
1	8017764A	M10*90 Stainless steel bulgy screw	6	
2	8006775A	M10*60 Stainless steel screw	3	
3	8013069A	M8*50 screw	1	
4	8017797A	ceiling light installation board	1	
5	8001621A	M8 Stainless steel nut	1	
6	8022822A	wire connector 2 port	2	
7	8012169A	M3*12 Stainless steel screw	2	
8	8015226B	24V 30VA transformer	1	
9	8009742A	M10 zinc-coated nut	12	
10	8001738A	M6*16 Stainless steel screw	4	
11	8017803B	ceiling adjustment board	1	
12	8017767A	M8*90 Stainless steel screw	1	
13	8001621A	M8 Stainless steel nut	2	
14	8014557A	8 feet ceiling light connection pillar 400 ( optional )	1	
	8014560A	9 feet ceiling light connection pillar 400 ( optional )	1	
	8014563A	10feet ceiling light connection pillar 400 ( optional )	1	
15	8017773A	ceiling light cover	1	
16	8017776A	ceiling light cover fixed ring	1	
17	8018190B	8018190B-DL-2 ceiling light power wire	1	