


Cabinet mount Whale LED Light

05KD01-00A

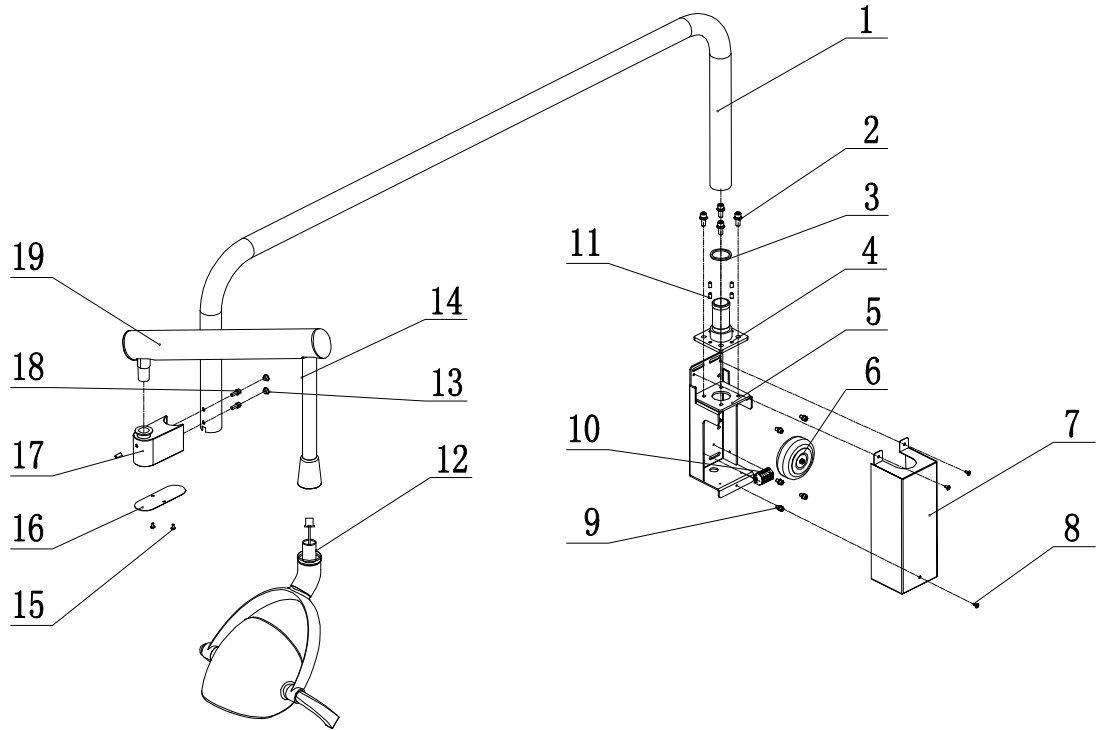

No.	Part number	Name	Number	Remark
1	8027082A	DL-5 light arm	1	
2	8020848A	M8*20 Stainless steel screw	4	
3	8027085A	DL-5φ42xφ35 flat pad	1	
4	8027076A	DL-5-light support pillar	1	
5	8027073A	DL-5 light power supply	1	
6	8015226B	24V 30VA transformer	1	
7	8027079A	DL-5 light power supply cover	1	
8	8016762A	M4*6 Stainless steel screw	3	
9	8024247A	M6*25 Stainless steel screw	4	
10	8005587A	Gray four-channel plastic terminal with fixing lug	2	
11	8013801A	M6*8 set bolt (black)	4	
12	8013552C	NICEDAY LED dental light	1	
13	8011491A	glass cuspidor silicone decoration cover	2	
14	8017794A	350 light installation rod	1	
15	8022177A	M4*12 Stainless steel screw	2	
16	8023533A	ceiling light balance arm fixed seat seal board	1	
17	8023530A	ceiling light balance arm fixed seat	1	
18	8001738A	M6*16 Stainless steel screw	2	