


# Classic100 Hanger

05ZS01-00A

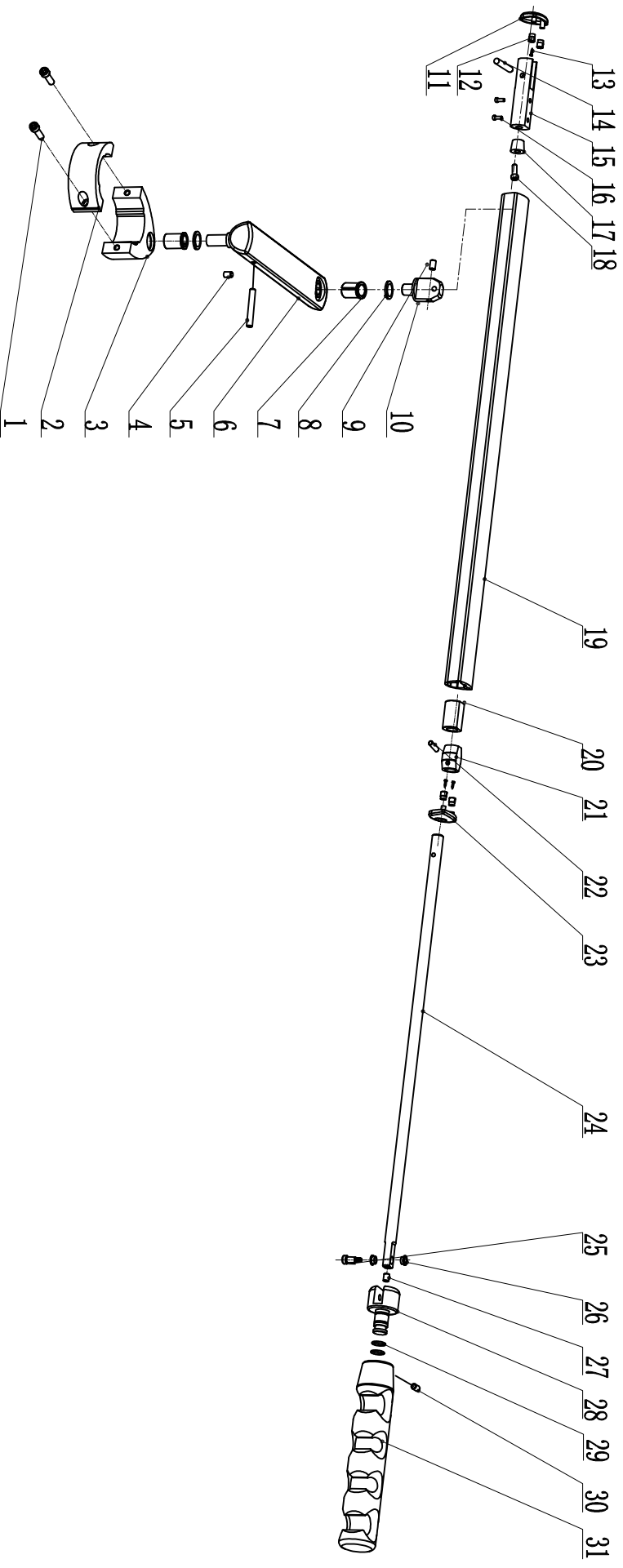

# Classic100 Hanger

05ZS01-00A

No.	Material No.	Name	Quantity	Remark
1	8001741A	M6*20 Stainless Steel Socket Head Screw	2	
2	8018817A	AS-5 Back Holder	1	
3	8018814A	AS-5 Front Holder	1	
4	8006037A	M6*8 Stainless Steel Socket Head Set Screw	2	
5	8018775A	AS-5 Limitation Pin	1	
6	8018823A	AS-5 Folding Arm	1	
7	8018772B	AS-5 Copper Bush	2	
8	8018811A	AS-5 Washer	2	
9	8010501A	M5*10 Stainless Steel Socket Head Screw	1	
10	8019987B	AS-5 Rotation Shaft II of Horizontal Arm	1	
11	8018808A	AS-5 Back Cover of Horizontal Arm	1	
12	8019393A	Rubber Plug	4	
13	8019591A	ST2.2*10 Stainless Steel Self-tapping Philips Screw	4	
14	8018784A	AS-5 Fixed Pin of Rotation Shaft	1	
15	8018781A	AS-5 Fixed Base of Rotation Shaft	1	
16	8010489A	M5*12 Stainless Steel Countersunk Head Philips Screw	2	
17	8018919A	15*12*8 Rubber Mat	1	
18	8001681A	M4*10 Stainless Steel Pan Head Philips Screw	1	
19	8018820C	AS-5 Horizontal Arm	1	
20	8018796B	AS-5 Locating Sleeve of Telescopic Pole	1	
21	8018790B	AS-5 Slide Block of Telescopic Pole	1	
22	8018793A	AS-5 Fixed Pin of Slide Block	1	
23	8018805A	AS-5 Front Cover of Horizontal Arm	1	
24	8019990B	AS-5 Horizontal Arm II	1	
25	8021841A	φ6*φ16*M5 Shoulder Screw	1	
26	8021829A	AS-5 Shaft Sleeve of Horizontal Arm	2	
27	8020032A	M6*10 Stainless Steel Fastening Screw	1	
28	8020014C	AS-5 Rotation Shaft of Hanger	1	
29	8017212A	φ12*φ1.5 O Ring	1	
30	8021145A	M6*5 Socket Head Set Screw	1	
31	8020008B	AS-5 Rotation Shaft of Hanger	1	