


Installation Manual of Beyond401 Delivery System

7. Water & Air diagrams


Figure 46 Water & Air diagrams

1. 1/8" Green, connect to water bottle
2. 1/4" Red, connect to water bottle
3. 1/4" Blue, connect to water bottle
4. 1/4" Black, connect to delivery head
5. 1/8" Red, onnect to delivery head

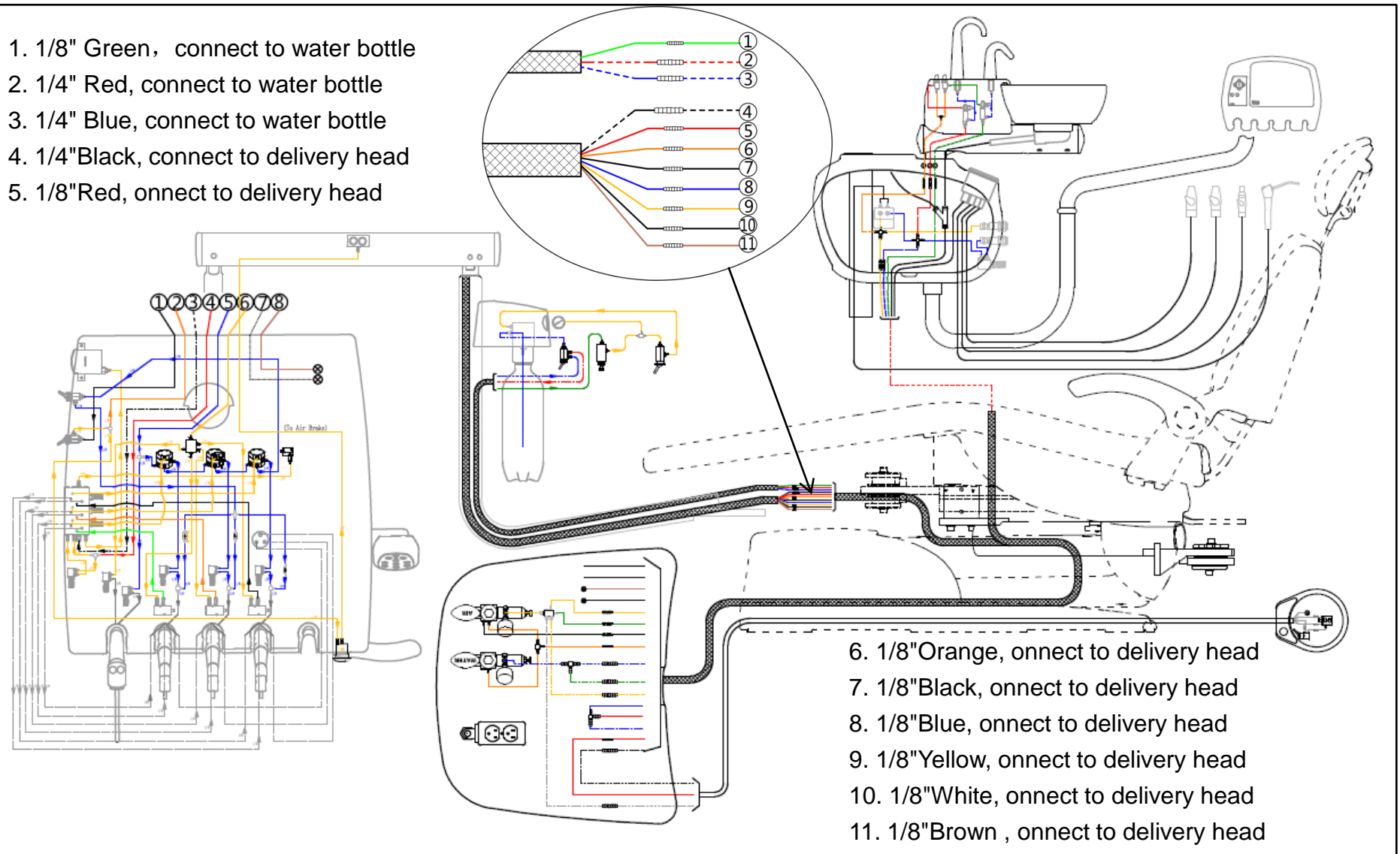

Installation Manual of Beyond401 Delivery System

Figure 47 Water & air diagrams of Beyond401 (delivery head)

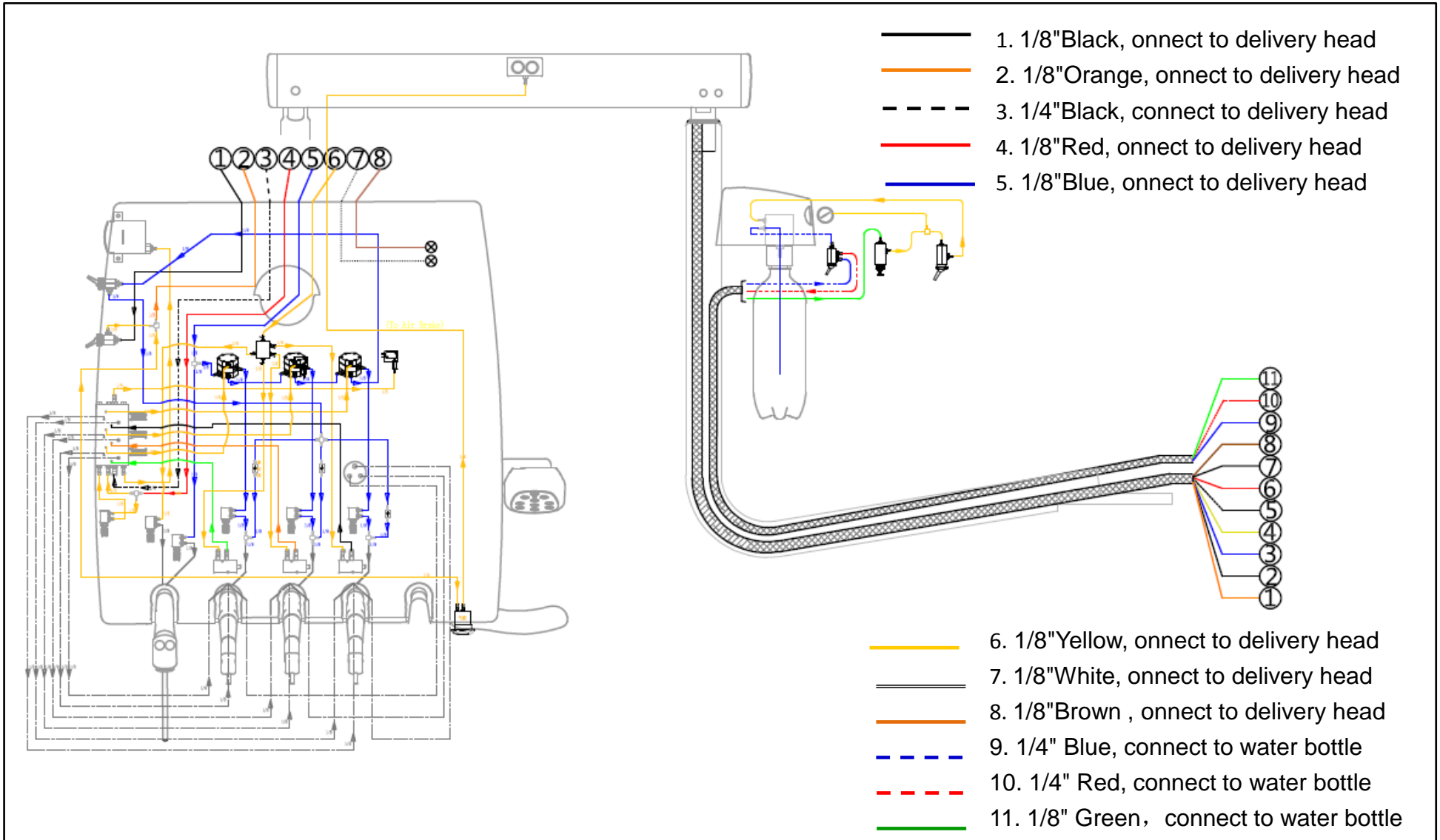

Installation Manual of Beyond401 Delivery System

Figure 48 Water & air diagrams of Beyond401 (water unit)

3. 1/8" yellow, connect to delivery system
4. 1/8" green, connect to delivery system
5. 1/8" black, connect to delivery system
6. 1/8" orange, connect to delivery system
7. 1/4" blue, connect to side box
8. 1/4" green, connect to side box
9. 1/4" Yellow, connect to side box
10. Drainage pipe, connect to cuspidor
11. Drainage pipe, connect to filter
- A. Pipe for foot control

